



天堰科技
TELLYES SCIENTIFIC



Lower Stump Bandaging Simulator

TYE1507

User Manual

TELLYES SCIENTIFIC INC.

About the Simulator

Represents the lower body of an adult male from below the xiphoid process to lower limbs (right leg amputated below the knee and left leg amputated at mid-thigh) with explicit anatomic landmarks including navel, anterior superior iliac spine, genitalia, buttocks cleft for training of bandaging.

Installation

1. Get the simulator out from the packing box and place it on a flat table, the height of which should be appropriate for operation.
2. The simulator is ready for use.

Instructions

1. The simulator can be used for training of bandaging skills including circular, oblique, spiral, reverse spiral, figure of eight (criss-cross) and recurrent bandaging etc.
2. Various bandaging methods for specific parts
Circular—thigh, shank
Oblique —thigh, shank
Spiral —thigh, trunk
Reverse spiral — shank
Figure of eight — knee
Recurrent — stump
Triangular bandage— lower abdomen, buttocks, lower limbs
Medical abdominal belt— abdomen

Storage

Tidy up the simulator after each use and keep in packing box for storage.

Maintenance and Care

1. Generally, dirt on the manikin surface can be cleaned with neutral soap and warm water.
2. Do not use colored disinfectant or detergent on the surface of the simulator. Avoid contact with ink, newspapers or other colored materials which may leave permanent stain on the simulator.
3. Treat the manikin gently as you would with a real patient, and misuse will shorten its service life.
4. To maintain good working status, clean the product after each use and do regular check.
5. Properly store the product in packing box away from sunlight after each use.

Specifications

1. Dimensions: 900×320×240mm
2. Product Weight: 6kg